

**i**fix

A NOVEL PATIENT STABILIZATION TECHNOLOGY



**i** INTERVENTIONAL  
systems

*SHAPING THE FUTURE  
OF PATIENT STABILIZATION*

## TECHNOLOGY



iFIX is a patient stabilization system comprised of both reusable and disposable components.

The disposable iFIX Fleece is attached to a reusable anchor product.



The unique micro anchors of the iFIX Patches and Adapters strongly secure the iFIX Fleece for efficient stabilization.



The comfortable stretchy fleece contours to the patient's anatomy like a second skin and secures the desired position.



Depending on your table's design, you can choose between the iFIX Patch that adheres to the table surface, the iFIX Slot Adapter that slides into the channels, or the iFIX Patch Adapter which connect to the fleece products.

# IN JUST 4 SIMPLE STEPS THE PATIENT GETS STABILIZED



1a




1b

1

Estimate the required length of iFIX Fleece and cut.



2a

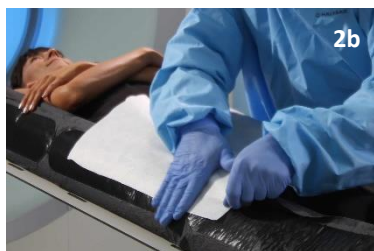

2b

2

On the opposite side of the table attach the iFIX Fleece to anchor product.

Anchor products:  
iFIX Patch, iFIX Patch Adapter,  
iFIX Slot Adapter



3a



3b

3

Pull iFIX Fleece tightly, up and over the patient.

Do not restrict patient's breathing.



4a



4b

4

Adhere the iFIX Fleece to the anchor product on the other side of the table and stabilize the patient.

If necessary, adjust iFIX Fleece to increase or decrease stabilization.

## BENEFITS

simple  
secure  
intuitive

SAFE

HYGENIC

disposable components lower risk for cross contamination

EFFICIENT

no cleaning time  
cost effective  
fast user training

i  
fix

head to toe  
solution

VERSATILE

MULTI  
MODALITY

supports several imaging modalities  
(CT, X-Ray, MRI, PET/SPECT, US)



Hip and lower spine stabilization



Hip and upper extremity stabilization



Head stabilization with the iFIX Head Mask



Head stabilization with head band



Lower extremity stabilization



Window for direct access

